

**Gulden Ophthalmics** *Instructions for Use: Universal Glasses Patch Part #14104 (Soccer Ball Design) and Part #14105 (Flower Design)*

---



**Gulden Ophthalmics**

225 Cadwalader Avenue  
Elkins Park, PA 19027-2020, USA

Web: [www.guldenophthalmics.com](http://www.guldenophthalmics.com)

Email: [info@guldenophthalmics.com](mailto:info@guldenophthalmics.com)

Phone: +1(215) 884-8105



**Qarad B.V.**

Flight Forum 40  
5657 DB Eindhoven  
The Netherlands



Insert the frame tip of the glasses through the front of the patch.

Slide the frame tip through the elastic loop.

Move patch around the frame and over-top the lens.

Open the glasses to reveal the peripheral blocking patch.

Cleaning: The eye patches can be gently hand-washed, reshaped, and air-dried.