

Instructions for Use

1. Remove Phak-i® from cylinder.
2. Turn the I-BORG™ over and gently rotate the circular ring to the horizontal position. This can be done by grabbing the white bar between the eyes and moving in a downward position.
3. Using your index finger and thumb, hold the circular ring and insert the base of the Phak-I on the top of the circular ring. Make sure that the bottom of the eye is positioned on ring with the arrow ↑ on the ring pointing up
4. To remove eye:

Turn the I-BORG™ over. CAUTION! Make sure you grab the Phak-i® from the base of the eye, not just the circular ring. Gently rotate circular ring in a downward position and remove the eye from the circular ring.

Attaching the bag:

Remove white cap from the bag and replace with the white cap with the luer lock adapter.

Take tubing out of bag and insert the white end of the tubing to the blue luer lock adapter on the I-BORG™. Insert the blue end of the tubing to the white cap on the bag.

