



Gulden Ophthalmics

225 Cadwalader Avenue

Elkins Park, PA 19027-2020, USA

Web: www.guldenophthalmics.com

Email: info@guldenophthalmics.com

Phone: +1(215) 884-8105



Qarad B.V.

Flight Forum 40

5657 DB Eindhoven

The Netherlands



1. Hold card approximately 14 inches from eye. If you use reading glasses or bifocals, wear them for this test.
2. Cover the left eye and look at the center white dot at all times.
3. Observe carefully for any distortion or waviness of the lines while staring at the center white dot.
4. Repeat the test by covering the right eye and viewing with the left eye.
5. If you notice any distortion, change or waviness in the lines while covering either eye, call your eye care practitioner immediately.