



Gulden Ophthalmics

225 Cadwalader Avenue

Elkins Park, PA 19027-2020, USA

Web: www.guldenophthalmics.com

Email: info@guldenophthalmics.com

Phone: +1(215) 884-8105



Qarad B.V.

Flight Forum 40

5657 DB Eindhoven

The Netherlands



This test was designed to test patients who are not responsive to the Worth Four Dot Test, notably children who do not recognize colors and also those with low color perception.

The objects are a red girl, a green elephant, and a white ball.

The three-battery flashlight has a much larger cover glass to accommodate the three Character Test. If suppression occurs, only two characters will be seen. With the red lens on the right eye, if the right eye is suppressed the girl will disappear. If the left eye is suppressed, the elephant will disappear.