

Gulden Ophthalmics *Instructions for Use: Magnetic Foreign Body Remover Cleaning Instruction*
Part #16108 (Ellis Style) and Part #16109 (Dix Style)



Gulden Ophthalmics

225 Cadwalader Avenue
Elkins Park, PA 19027-2020, USA

Web: www.guldenophthalmics.com

Email: info@guldenophthalmics.com

Phone: +1(215) 884-8105



Qarad B.V.

Flight Forum 40
5657 DB Eindhoven
The Netherlands



Because Magnetic Foreign Body Removers come into contact with the cornea and tear film, they must be properly disinfected to prevent cross-infection with HIV and other infectious pathogens. The Centers for Disease Control and Prevention (CDC) recommends that all eye contact devices be disinfected by a 5-10 minute soak in one of following disinfectants: 3% hydrogen peroxide or 70% isopropyl alcohol.